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IVIP® A2

CODAN products IVIP® A2

with Anti-Free-Flow clamp (AFF clamp) suitable for use with the following compatible pump :
arcomed Volumed® μ VP7000

Use of product / Notes on safe use

- Please observe any further instructions for use that are enclosed with this medical device or the instructions for use of any compatible medical devices used in parallel.
- Before use, check that the pump calibration matches the tube material of the product.
- If necessary, move the AFF clamp of the product to the required position on the tube.
- Close the AFF clamp first. Open the door of the infusion pump and insert the closed AFF clamp into the clamp holder according to the direction / marking indicated in the infusion pump.
- Place the tubing straight over the delivery element and into the tubing guidance channels on both side of the infusion pump. Pay attention to the flow direction marked by the arrow in the infusion pump.
- Ensure that the tubing and AFF clamp are properly inserted before closing the door of the infusion pump.
- If applicable, attach the drip detector of the pump to the drip chamber of the product.
- The AFF clamps closes automatically when the infusion pump door is open.
- Before removing the tubing and AFF clamp from the infusion pump, stop the running infusion, close the roller clamp and if necessary disconnect the patient connection port from the patient access to prevent free flow.
- The product can also be used without using an infusion pump and thus without function of the Anti-Free-Flow clamp under gravity and with a pressure cuff with generated pressures of up to 50 kPa (0.5 bar).

Product compatibility

The product is compatible with the following volumetric infusion pump :
arcomed Volumed® μ VP7000

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