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 **0123**

<p>en</p>	<p>10.1230.00 / 2020-10</p>
<p>Instructions for use for the sterile single packed and integrated SWAN-LOCK® / SWAN-LOCK® RED (hereafter stated as SWAN-LOCK®)</p>	
<p>Normal diligent hygiene practice shall be observed at all times during handling of SWAN-LOCK®.</p>	
<p>For the sterile single packed SWAN-LOCK® follow steps 1. - 5. and for integrated SWAN-LOCK® steps 3. - 5. of the instructions for use.</p>	
<ol style="list-style-type: none">Take the SWAN-LOCK® out of its packaging. Connect the Male Luer-Lock connection of SWAN-LOCK® with an intravenous catheter or other Female Luer-Lock connector. The top surface of the SWAN-LOCK® shall be swabbed with disinfectant prior to each use. Isopropyl alcohol, iodine-containing alcohol or ethyl alcohol 70 % or isopropyl alcohol 70 % in combination with chlorhexidine gluconate 2 % can be used for disinfection. The exposure time has to amount to a minimum of 30 seconds. It is to be ensured that a connection is only made when completely dried. Attach the Male Luer taper of an administration set or single-use syringe to the Female Luer-Lock end of the SWAN-LOCK®. This will open the silicone valve. A slight push and twist action is recommended when attaching Luer slip adapters to the SWAN-LOCK® in order to ensure a tight fit and the correct functioning of the valve. Do not use Luer adapters with extra-long tapers as they might cause a malfunction or even damage the valve. The SWAN-LOCK® shall be flushed after addition of medication or the withdrawal of fluids. This will eliminate residues, act as a prophylactic against incompatibilities and prevent malfunctioning due to internal dryness creating excessive friction.	
<p>Note:</p> <p>The SWAN-LOCK® can be used for up to 140 activations during a permitted application time according to the national directives for prevention of catheter related blood stream infections. Whilst the SWAN-LOCK® will readily accept Luer Slip fittings, it is recommended that Luer-Lock fittings are used for additional security where the patient will be unattended by the caregiver.</p> <p>Do not use needles with the SWAN-LOCK®.</p>	

<p>es</p>	<p>10.1230.00 / 2020-10</p>
-----------	-----------------------------

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