
SWAN-LOCK® / SWAN-LOCK® RED

CODAN

- en** Swabable needle-free adapter
Instructions for use
- de** Desinfizierbarer nadelfreier Adapter
Gebrauchsanweisung
- cs** Otíratelný bezjehlový adaptér
Návod k použití
- da** Desinficerbar nålfri adapter
Brugsanvisning
- es** Adaptador sin aguja desinfectable
Instrucciones de uso
- fr** Adaptateur sans aiguille désinfectable
Notice d'utilisation
- it** Adattatore senz' ago disinfettabile
Istruzioni per l'uso
- nl** Desinfecteerbare naaldloze adapter
Gebruiksaanwijzing
- no** Desinfiserbar nålfri kobling
Brukerveiledning
- pl** Adapter bezigłowy do szybkiej dezynfekcji
Instrukcja używania
- pt** Adaptador sem agulha desinfectável
Instruções de utilização
- sv** Desinficerbar nålfri adapter
Bruksanvisning

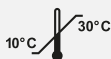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CE 0123

**Instructions for use
for the sterile single packed and integrated SWAN-LOCK® / SWAN-LOCK® RED
(hereafter stated as SWAN-LOCK®)**

Normal diligent hygiene practice shall be observed at all times during handling of SWAN-LOCK®.

For the sterile single packed SWAN-LOCK® follow steps 1. - 5. and for integrated SWAN-LOCK® steps 3. - 5. of the instructions for use.

1. Take the SWAN-LOCK® out of its packaging.
2. Connect the Male Luer-Lock connection of SWAN-LOCK® with an intravenous catheter or other Female Luer-Lock connector.
3. The top surface of the SWAN-LOCK® shall be swabbed with disinfectant prior to each use. Isopropyl alcohol, iodine-containing alcohol or ethyl alcohol 70 % or isopropyl alcohol 70 % in combination with chlorhexidine gluconate 2 % can be used for disinfection. The exposure time has to amount to a minimum of 30 seconds. It is to be ensured that a connection is only made when completely dried.
4. Attach the Male Luer taper of an administration set or single-use syringe to the Female Luer-Lock end of the SWAN-LOCK®. This will open the silicone valve. A slight push and twist action is recommended when attaching Luer slip adapters to the SWAN-LOCK® in order to ensure a tight fit and the correct functioning of the valve. Do not use Luer adapters with extra-long tapers as they might cause a malfunction or even damage the valve.
5. The SWAN-LOCK® shall be flushed after addition of medication or the withdrawal of fluids. This will eliminate residues, act as a prophylactic against incompatibilities and prevent malfunctioning due to internal dryness creating excessive friction.

Note:

The SWAN-LOCK® can be used for up to 140 activations during a permitted application time according to the national directives for prevention of catheter related blood stream infections. Whilst the SWAN-LOCK® will readily accept Luer Slip fittings, it is recommended that Luer-Lock fittings are used for additional security where the patient will be unattended by the caregiver.

Do not use needles with the SWAN-LOCK®.

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