nl

10.1267.00 / 2022-10

CODAN Guard® - Speciaal AirStop filtermembraan

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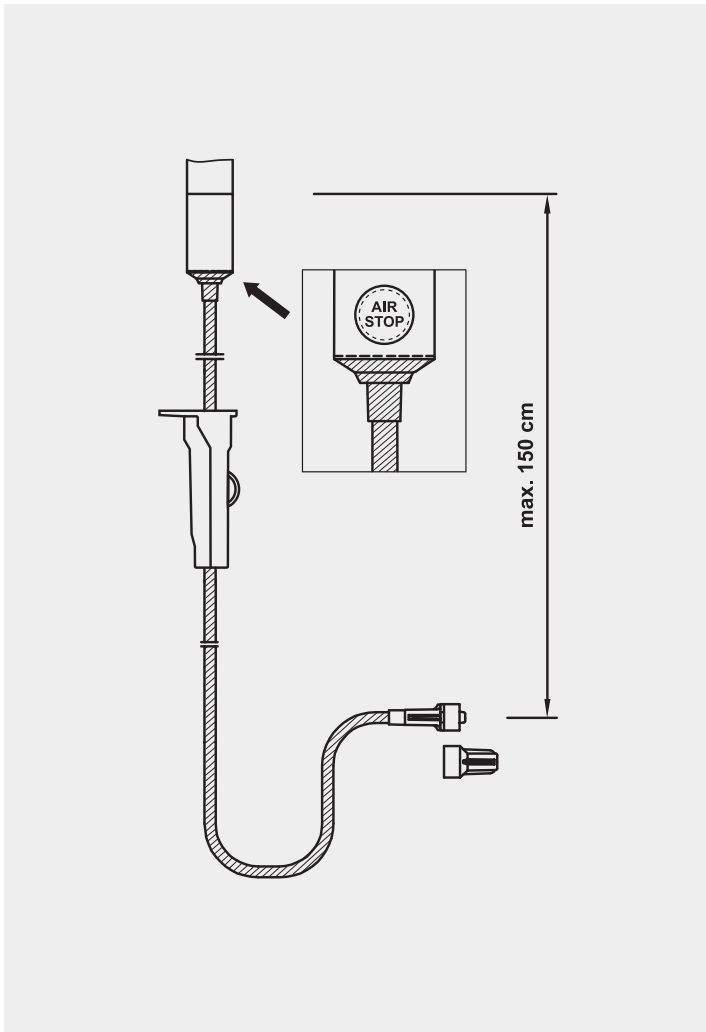
en
10.1267.00 / 2022-10
CODAN Guard® - Special AirStop filter membrane with pore size 8–10 µm integrated into the drip chamber for infusion therapy.
Intended purpose The AirStop filter membrane prevents the infusion set from running empty below the drip chamber and thus prevents air from entering the system.
Clinical benefits The special AirStop filter membrane additionally enables the risk of air embolism to be reduced.
Notes on safe use of product • Please consider all further instructions for use enclosed with this medical device or the instructions for use of compatible medical devices used in parallel. • The administering of certain pharmaceuticals can lead to a reduced flow rate due to the pore size 8–10 µm of the AirStop filter membrane. Please always observe the Summary of Product Characteristics of the pharmaceutical manufacturer. • The AirStop feature is activated by contact of the filter membrane with liquid for at least 60 seconds. • The AirStop function may be impaired if a water column of 150 cm is exceeded. • The AirStop function is ineffective when used with pumps.
For further information (e.g. indications, contraindications, etc.), please observe the accompanying instructions for use of the product.
Valid for This supplement to the instructions for use is valid for infusion sets with special AirStop filter membrane containing CODAN Guard® in the product name.

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