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STERILE EO MD

Transfusion set

suitable for use with compatible pumps in particular for neonatology/paediatrics applicable

Intended purpose

Transfer of blood and blood components by intravenous infusion. Isotonic electrolyte solution can additionally be used for priming and rinsing the product.

Indications

Diseases and conditions requiring intravenous infusion of blood and blood components.

Contraindications

The product is not designed or approved for:

- Home use
- Use by unqualified, untrained staff

Self-application is not permitted and is not considered intended use.

The product is not intended for the transfer of:

- Liquids, pharmaceuticals or parenteral nutrition solutions except those specified under intended use
- Enteral nutrition solutions

The current Summary of Product Characteristics of the pharmaceutical manufacturers for the transfer of blood/blood components/electrolyte solutions with regard to incompatibilities (interaction material/ pharmaceutical and interaction of different pharmaceuticals) and contraindications shall be observed. Special attention has to be paid to vulnerable patient groups such as children and adolescents as well as pregnant or lactating women.

Patient target group

from birth/ from birth weight (including premature babies). The product can be used for any patient populations with consideration given to the adequacy of vascular anatomy, appropriateness of the solution being infused, and the duration of therapy.

User

Qualified medical personnel only. For use in hospitals and specialised practices with conventional environmental and hygiene conditions.

Clinical benefits

Life-saving and/or health-improving therapeutic benefits to patients. The transfusion of blood and blood components carries implicit risks and should therefore only be carried out if it improves patient's health.

General safety instructions

• The product is a medical device. • The instructions for use shall be observed before using the product. • If the sterile packaging is damaged or unintentionally opened before use, the product has to be completely disposed of and replaced by a new one. • The product is intended for single use. Resterilisation as well as reuse are not permitted. Reuse can cause infections. • In the case of malfunctions (e.g. leakage, detached protective caps) or performance changes, the product shall not be used on the patient, and has to be replaced. The defective product is to be provided to the manufacturer in compliance with measures for safe transport. • Due to the sterilisation method, the product can contain traces of ethylene oxide and its residual products. In case of known allergies to these substances, refrain from using the product. • Prior to the transfer of blood and blood components, the qualified medical personnel shall conduct a patient-specific benefit-risk assessment. • Furthermore, the international and national guidelines for the transfer of blood and blood components shall be taken into account. In particular, blood group compatibility between donor and recipient as well as the EN ISO 1135-5 recommendation not to transfer platelet components under pressure shall be considered. • The function of the product shall be checked by qualified medical personnel at regular intervals during use. • Always observe the instructions for use from the pharmaceutical manufacturers regarding incompatibilities and the current Summary of Product Characteristics. • Generally accepted hygiene measures and working methods have to be observed at all times. • Use of protective medical gloves is recommended to prevent any infections of users.

Notes on safe use of product

• The product is designed for use with compatible pressure infusion pumps with generated pressures up to 200 kPa (2 bar). • Notes on use with pumps: Prior to pump operation, the pump has to be calibrated to the used syringe. Always observe the instructions for use from the pump manufacturer. Keep closure clamp closed during pump operation! • The

product cannot be used under gravity. • Before using the product, compare the permissible fluid intake of the patient with the suitability of the product. • A complete wetting of the fluid filter in the drip chamber with isotonic electrolyte solution is recommended prior to the administration of blood and blood components. • Prior to use, the product shall be checked for damage, tightness and any misconnections. • Existing closure plugs have to be checked on tightness and retightened if necessary. • Priming of the product with FlowStop protection cap: carefully prime the product via syringe with a maximum pressure of 0.15 bar. Do not exert manual pressure on the infusion/blood container. • Prior to and during use of the product, all connection ports shall be disinfected before and after each connection/disconnection. Isopropyl alcohol, iodine-containing alcohol or ethyl alcohol 70 % or isopropyl alcohol 70 % in combination with chlorhexidine gluconate 2 % can be used for disinfection. Please observe that the exposure time has to amount to a minimum of 30 seconds. It is to be ensured that a connection is only made when completely dried. • Excessive forces on Luer-Lock connections can lead to stress cracks. • If equipped with an orange protective cap (vented), replace it with a closure plug (non-vented).

Use of product

Please also observe the exemplary product illustration(s) as well as further information/images for product use on the product packaging.

Preparation of the transfusion set (cf. exemplary product figure(s)):

1. Check if the sterile packaging is damaged or unintentionally opened.
 2. Remove the product from the sterile packaging and check for possible damage.
 3. Close the closure clamp (4).
 4. Remove the protective cap and insert the non-vented closure-piercing device (1) into the infusion/blood bag with a rotating movement. Please ensure that the wall of the blood bag remains intact during piercing.
 5. Fill the drip chamber (3) up to the level ring (2).
 6. Connect a Male Luer-Lock syringe to the needle-free injection site (6).
 7. Open the closure clamp (4).
 8. Aspirate approximately 5 ml of the blood product into the syringe to prime that part of the system.
 9. Hold the syringe vertically to eliminate the air through the remaining set first and then carefully prime the remaining system completely, so that the transfusion device is applied without any air inclusions. The integrated check valve (5) prevents any backflow to the blood bag.
 10. If there is any time interruption between preparation and administration, please close the closure clamp (4).
- Administration to the patient:
11. If there was a time interruption between preparation and administration and the closure clamp (4) is closed, please open it now.
 12. Aspirate the required quantity of blood product into the syringe.
 13. Close the closure clamp (4).
 14. Insert the syringe into a compatible syringe pump.
 15. Connect the patient connection port (9) to the patient access.
 16. Start the syringe pump according to the required medication and patient parameters. Closure clamp (4) has always to be closed during syringe pump administration.
 17. Check the transfusion at regular intervals.

Product compatibility

The closure-piercing device complies with EN ISO 1135-5 and can be connected with all compatible infusion/blood containers. All Luer and Luer-Lock connectors are compatible according to EN 1707 and EN 20594-1. Compatible with syringe pumps with a pressure alarm limit not exceeding 2.0 bar.

Residual risks

Due to leakage or malfunction, biological, physical or chemical contaminations, non-sterility, misconnections, degradations or of damage to blood components, changed volume flow, chemical or physical interactions, the following possible health risks can occur:

- Infections
- Skin irritations/skin injuries
- Air embolism

- General degradation of health status
- Toxic effects

The patient or their legally responsible person should be informed about potential health risks prior to treatment.

Period of use

The international and national guidelines for the transfer of blood and blood components shall be taken into account. A maximum period of use of 12 hours should not be exceeded. Taking into account the individual benefit-risk ratio of each patient, a longer period of use can be considered by the physician.

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Product disposal

The product shall be disposed of in accordance with the medical regulations for the disposal of contaminated medical waste. Insert the closure-piercing device into the notch on the roller clamp to avoid any injuries. Protective medical gloves have to be worn during disposal.

Notification of serious incidents

All serious incidents occurring in connection with the product shall be reported to the manufacturer and the corresponding national reporting authority without delay. Defective products are to be provided to the manufacturer in compliance with measures for safe transport.

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