


# CODAN Xtrans®

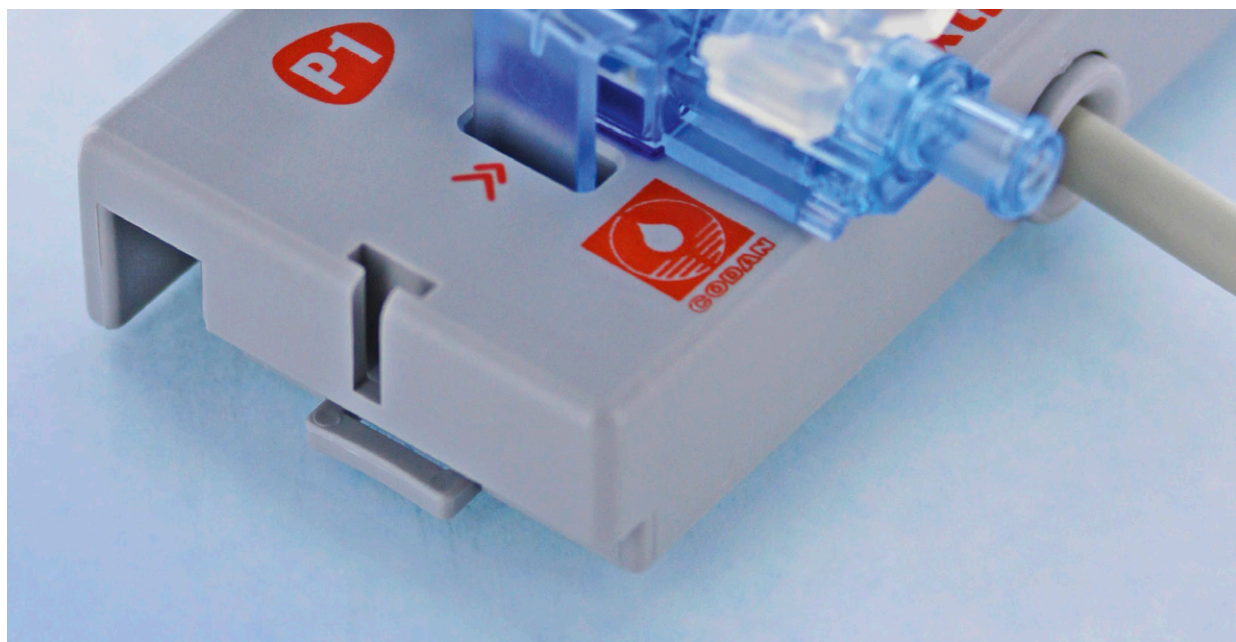

The intelligent  
pressure monitoring system



# CODAN Xtrans®

## Intelligent solutions make technology simple

### Amazingly simple to use


The Xtrans® pressure transducer is simply clicked into the contact board, and is ready for operation! The electrical connection to the monitor is established simultaneously. Simplicity in use combined with ergonomic design - that's what distinguishes the Xtrans® system.

### Unique, reliable contact

In addition to the simple, rapid connection of the Xtrans® pressure transducer to the contact board, Xtrans® also guarantees reliable, secure contact. The special contact design is resistant to wetting, both before and during priming.

### Technology + design = reliability

The Xtrans® pressure transducer is equipped with an ultra-sensitive sensor. To prevent any distortion of the pressure measurement, the sensor is located directly beneath the flow channel. The laminar flow channel ensures optimal priming and flushing of the system. After all, only a completely vented system can deliver the key requirement: precise pressure monitoring.

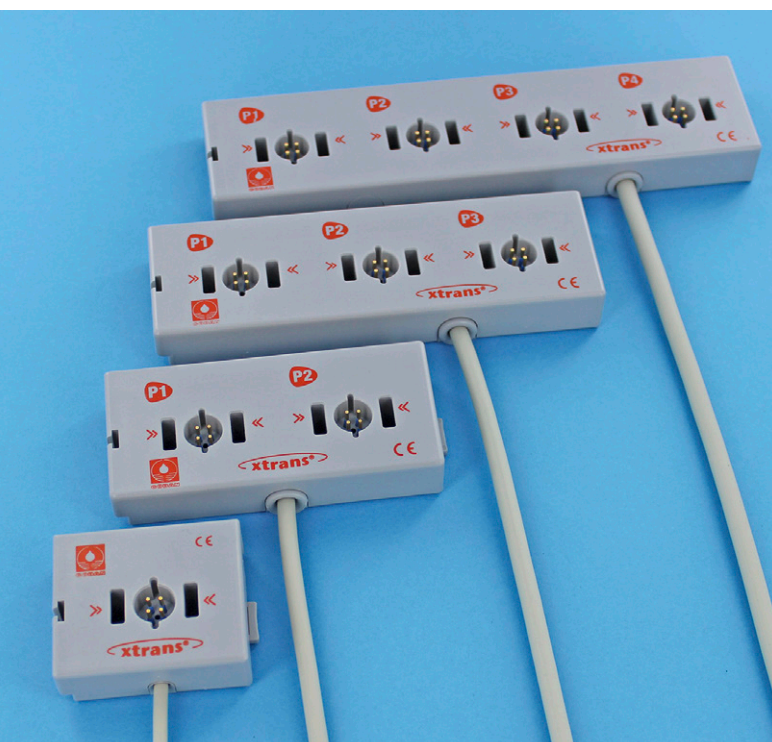

### Cost-effective working with Xtrans®

The Xtrans® system is designed to optimise material usage. There are absolutely no superfluous components: no transducer cable and no additional retaining plates. As a result, Xtrans® weighs in at just 12 grams.



# The system

**The ground-breaking innovation in invasive blood pressure monitoring - simple, safe, cost-effective.**


## **No compromises when it comes to precision**

The Xtrans® system includes both disposable and reusable components.

The disposable components ensure sterility and a precise signal in all pressure ranges.

The reusable components are made of very robust, high-quality materials, ensuring a long service life for the Xtrans® system.

**Xtrans® - that means reusable components where it makes sense.**

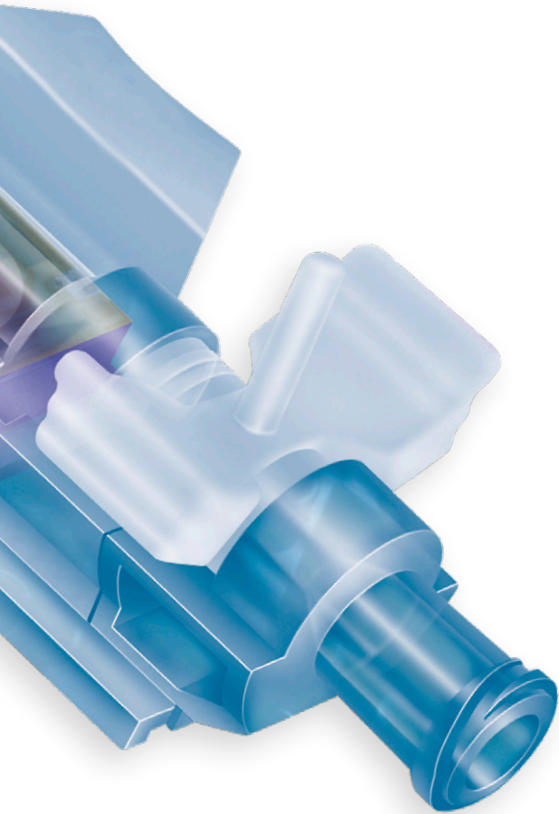

## **Professional system design**

Contact boards are available for 1 to 4 transducers, as required for single or multiple pressure monitoring. No need to connect together individual boards – that means robust, compact units.

Irrespective of whether single or multiple port contact boards are used – there is only one integrated cable leading to the monitor. No individual interface cables.

The cable is fitted with a matching connector for the monitor connector which has 1-4 pigtailed connectors for the monitor modules.

The monitor connector can be disconnected from the cable, and then remains in the monitor permanently. This makes it easy to switch between different monitor models.



## **Contact board xxs for special applications**

The contact board xxs is for applications where a pressure monitoring system needs to be light and compact - ideal for attaching to the patient's arm.

# CODAN Worldwide



CODAN is known internationally as a manufacturer and supplier of medical transfer systems. The CODAN Companies have more than 1500 employees around the world.

The name CODAN is synonymous with reliability, quality and precision based on the know-how and experience gained from more than 60 years of research and development. Company-owned production facilities and sales companies around the world are a guarantee for efficient production, a tight-knit sales network and a first-class service for customers in the healthcare sector.

## CODAN Companies

CODAN Medizinische Geräte GmbH · Deutschland  
CODAN pvb Critical Care GmbH · Deutschland  
CODAN pvb Medical GmbH · Deutschland  
CODAN 11, S.A. · Portugal  
CODAN US Corporation · California · USA  
CODAN Inc. · California · USA  
CODAN NORGE AS · Norge  
CODAN TRIPLUS AB · Sverige  
CODAN Limited · Great Britain  
CODAN FRANCE Sarl · France  
CODAN Medical AG · Schweiz  
CODAN ARGUS AG · Schweiz  
CODAN BV · Nederland  
CODAN s.r.l. · Italia  
CODAN Medical GmbH · Österreich  
CODAN Medical ApS · Danmark  
CODAN DEHA ApS · Danmark  
CODAN MEDITECH s.r.o. · Česká republika

### CODAN Product range

- Infusion sets
- Transfusion sets
- Extension lines and manifold connectors
- Infusion and transfusion accessories
- Infusion filters and filter systems
- Neonatology/Paediatric products
- Withdrawal, preparation and administration systems
- CODAN CYTO®
- Chemoprotect® products
- Single use syringes
- Invasive blood pressure monitoring systems
- Infusion pumps
- Other CODAN Products

Compliance of the established quality management systems with the provisions of EN ISO 13485 and the Council Directive 93/42/EEC has been certified by the relevant, competent notified bodies:

### TÜV SÜD Product Service GmbH

CODAN Medizinische Geräte GmbH  
23738 Lensahn, Germany

CODAN pvb Critical Care GmbH  
85661 Forstinning, Germany

CODAN US Corporation · Santa Ana, CA 92704, USA

CODAN ARGUS AG · 6340 Baar, Switzerland

### Presafe Denmark A/S

CODAN Medical ApS · 4970 Rødby, Denmark



### Manufacturer

CODAN pvb Critical Care GmbH  
Römerstraße 18 · 85661 Forstinning, Germany  
Tel. +49 (0) 81 21 · 98 02 0  
codan@codanpvbcc.de · www.codancompanies.com

**The decisive connection**