Berner SpillKit XP for CODAN

User Information

Emergency cleaning kit for spilled cytotoxic drugs

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Language

In the following you can find the user information for the Berner SpillKit XP for CODAN latexfree in size L in 12 languages.

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User information

Read the user information carefully. Disregarding the user information can impair the performance of the SpillKit components or be harmful to the health of the user and endanger the environment.

Contents

The Berner SpillKit XP for CODAN is suitable for the safe disposal of small quantities of spilled CMR drugs (e.g. cytostatics) and contains:

1 Protective gown cleo® saphir Chemoprotect®, 2 pairs of Berner Manu Prene XP cytostatic protective gloves, 1 pair of green uvex Profastrong chemical protective overgloves, 1 pair of protective overshoes cleo® saphir Chemoprotect®, 1 pair of safety goggles, 1 FFP-3 respiratory mask.

Tools: 1 ChemoSorb Pad, 6 Berner Isysoft wipes, 1 Green Z® powder, 1 bottle of distilled water, 1 tweezer, 1 sweeper, 1 scoop, 1 marker pen, 2 warning signs, 1 barrier tape, 1 insertion bag, 1 waste bag (hazardous waste), 1 disposal box, 1 cable tie, 1 accident report (only valid for Germany).

CE marking for PPE included in the SpillKit

The personal protective equipment (PPE) contained in the SpillKit is certified in accordance with PPE Regulation (EU) 2016/425 category II + III. You can find the declarations of conformity and further information on PPE on our website: www.berner-safety.de. Please note: A general declaration of conformity for the SpillKit cannot be issued due to the composition of the SpillKit (PPE and tools without CE marking).

Usage & disposal

The Berner SpillKit XP for CODAN is to be used only once to clean up spilled cytostatic drugs. After use, all components of the Berner SpillKit are to be classified as waste requiring special monitoring (waste code: 18 01 08* in accordance with 2000/532/EG).

Storage, transport and shelf life

Dark (keep away from direct UV and sunlight); cool (+5 to +40°C); dry; no contact with pointed or sharp objects. Fragile. Shelf life Berner SpillKit XP for CODAN: 3 years from date of manufacture.

Instructions for use

In the event of accidental cytostatic contamination, follow the instructions below:

Step 1: Keep calm!

Treat personal contamination before material contamination. In case of skin contact, rinse immediately with plenty of running cold water, do not use soap. After eye contact, immediately flush with plenty of water or isotonic saline solution for at least 10 minutes. Take off contaminated clothing immediately. If possible, you should bring in a second (auxiliary) person. Please note that decontamination should only be carried out by trained personnel.

Step 2: Observe user information/operating instructions!

The brief user guide should be kept close to the workplace. Carry out the individual steps as you have trained in your annual instruction.

Step 3: Don PPE!

First protect the people involved, put on the appropriate PPE and also ensure adequate protection for the second, helping person!

When putting on protective clothing, observe the correct sequence:

1. Put on the respirator. Hold the mask in your hand and let the headgear hang down. Bring the first strap up to the neck, now place the mask under the chin and pull the 2nd strap over the head. Adjust the nose clip to fit the nose. Cover the half mask with both hands and breathe in and out forcefully, the mask will move. If air escapes at the edge of the mask, adjust the mask and adjust the nose clip to fit your nose. Please note that beard and facial hair as well as scars can affect the fit of the mask. **2.** Don the goggles. The goggles can be worn over prescription glasses. **3.** Put on the cytostatic protective gown **4.** Put on the brown cytostatic protective gloves and pull the knitted cuff of the gown over the cuff of the brown gloves. **5.** Put on the protective over shoes. **6.** Now pull the green protective glove over the brown cytostatic protective glove, the cuff over the knitted cuff of the gown.

Step 4: Cordon off and mark the accident site!

Secure the accident site to prevent the substance from spreading. Mark the accident site generously with the enclosed pen and cordon off the affected area with the warning signs and barrier tape. Prevent turbulence: switch off the HVAC-System, close windows and doors and prevent pedestrian traffic. In the event of a spill in-side a safety cabinet, turn on the ventilation or leave it running.

Step 5: Note the form of disposal and cleaning direction!

Please note that the gloves usually do not offer sufficient protection against cuts! Therefore, use tweezer or a sweeper for glass and sharp-edged fragments. If glass with liquid cytostatic agents is broken, we recommend first sprinkling the Green Z® powder onto the liquid and then picking up these objects using the sweeper, tweezer and scoop.

In addition, the ChemoSorb pad, ISYSOFT wipes and sweeper can be used to absorb the liquid from the breakage. For powdered contamination, use the distilled water and ISYSOFT wipes: place the dampened wipes over the spill. The cleaning direction is always from the outside to the inside! Do not use hand brushes or similar, as this causes turbulence.

Step 6: Place the removed contamination in the disposal box!

Pick up glass and sharp fragments with the tweezer and place them in the box. Disposal of the pads and wipes in the box using the sweeper and scoop. Always carry out at least 3 cleaning stages, each with an alkaline cleaner (≥ pH 11) and isopropanol. Place the contaminated items, auxiliary materials and cleaning utensils in the disposal box and close the lid tightly.

Step 7: Put PPE and waste in the white hazardous waste bag!

Now put the closed disposal box and all materials in the white hazardous waste bag. Take off your protective clothing and put it in the white hazardous waste bag as well – be sure to do this in the correct order: **1.** over shoes **2.** green protective gloves **3.** protective gown **4.** brown cytostatic protective gloves **5.** protective goggles **6.** respiratory mask. Now close the white hazardous waste-bag with the cable tie. Observe the disposal instructions.

Step 8: Shower! Shower thoroughly.

Step 9: Doctor - Consult a doctor if necessary, especially if you have been personally contaminated.

Step 10: Accident Report: Write a detailed accident report.

You can use the enclosed accident report form (only for Germany). Conduct a debriefing and/or retraining after an accident, if necessary.

Be sure to get a new SpillKit.

Protective gown cleo® saphir Chemoprotect®, Size L

Type-tested and certified according to PPE regulation (EU) 2016/425, based on DIN EN 14605:2005 +A1:2009; EN 14126:2003. Notified body 2797.

Area of application: Type tested and certified as complex PPE (category III); chemical protective type PB [4], protective clothing against infection type PB [4]-B; partial body protection. Protective gown for handling CMR drugs (e.g. cytostatic and virostatic agents) and biological agents (e.g. bacterial and viruses). No guarantee for all CMR medicines or chemicals! Always wear with the coated side on the outside and the seam pointing downwards. Keep away from open flames and heat sources. Before use: Check for any damage! Do not use damaged gowns. Detailed data on mechanical and chemical protection as well as the declaration of conformity can be found on our website: www.berner-safety.de.

Protective overshoes cleo® saphir Chemoprotect®, Size uni

Type-tested and certified according to PPE regulation (EU) 2016/425 as complex PPE (category III); based on DIN EN 14605:2005 +A1:2009; EN 14126:2003. Notified body 2797.

Area of application: Type-tested and certified as complex PPE (category III); chemical protective clothing type PB [4], protective clothing against infection type PB [4]-B; partial body protection. Protective overshoe for handling CMR drugs (e.g. cytostatic and virostatic agents) and biological agents (e.g. bacterial and viruses). No guarantee for all CMR medicines or chemicals! Always wear with the coated side on the outside and the seam pointing downwards. Keep away from open flames and heat sources. Before use: Check for any damage! Do not use damaged overshoes! Detailed data on mechanical and chemical protection as well as the declaration of conformity can be found on our website: www.berner-safety.de.

Protective gloves Berner Manu Prene XP

Type-tested and certified according to PSA regulation (EU) 2016/425 (category III); Type A. The type examination carried out is based on EN ISO 374-1:2016 + A1:2018 Type A; EN16523-1:2015+ A1:2018, EN374-2:2019, EN374-4:2019; EN374-5:2016; EN 21420:2020; Notified body 2797.

Area of application: Protective gloves for handling CMR drugs and biological agents. No guarantee for all CMR medicines or chemicals! Before use: Check for damage! Do not use damaged protective gloves! Detailed data on mechanical and chemical protection as well as the declaration of conformity can be found on our website: www.berner-safety.de.

Protective gloves uvex Profastrong NF33, size 9

Certified according to PPE regulation (EU) 2016/425 for complex PPE category III based on EN 420:2003+A1:2009, EN 374-5:2016, EN 374-1:2016, EN 388:2016. Notified Body No. 2777.

Area of application: Type A chemical protective gloves; anatomically shaped; Protective properties: Good resistance to acids, solvents, oils and fats. No guarantee for all chemicals! Area of application: Glove to be worn over the cytostatic protective glove. Before use: Check for damage! Do not use damaged gloves!

FFP3 respiratory folding mask BLS Zer0 33

Filtering half mask with high filter performance (> 99%), tight fit and no leaks

Area of application: Protection against solid and liquid particles (also against nanoparticles up to 0.001 µm), e.g. for handling CMR drugs. Before use: Check for damage! Do not use damaged masks! Limitation: Laboratory test values can deviate significantly from the values achieved in practice, which can lead to longer or shorter holding times. Beard or facial hair as well as scars on the face can affect the seal of the mask and thus the protective effect. Certified

according to PPE regulation (EU) 2016/425 for complex PPE category III based on EN 149:2001+A1:2009. Type testing, certification and control measures by the notified body 2008. You can find more information and the declaration of conformity on our website: www.berner-safety.de.

Safety goggles uvex 9405

Safety goggles; anti-fog; can also be worn over prescription glasses.

Application range: Protection against mechanical risks, liquids and coarse dust. Protective property: Not suitable for high speed particles, laser beams, temp. over 55°C, arc flash, molten metals, infectious substances or organisms. Before use: Check for damage, do not use glasses where the marking is missing or illegible. CE marking according to PPE Regulation (EU) 2016/425 for complex PPE Category II, based on EN 166:2001. Type examination and certification by the notified body 0196.

Tools

ChemoSorb pads	For absorbing and binding liquids
ISYSOFT wipes	Extremely absorbent wipes, particularly suitable for absorbing spilled liquids.
Marker pen	Colour variable
Warning sign	Cardboard display, size: 297 x 210 mm (opened), warnings in German and English.
Scoop	Material: Plastic, size: 320 x 210 mm, Nitro- fest, bead back, toothless.
Sweeper	Material: plastic, size 135 x 100 mm
Bottle, distilled water	Plastic bottle, 500ml distilled water.
Insertion bag	Material: polyethylene size: 576 x 1000 mm, thickness: 50 µm Colour: transparent Weight: 46 g/m ² ± 5 g.
Hazardous waste bag	Polyethylene with imprint: "Cytotoxic and cytostatic drugs", colour white, weight: 127 g ± 5 g
Cable ties	Plastic, size 200 x 3.6 mm, color: white
Barrier tape	To secure the affected area, 25m, color: red- white
Tweezer	To collect broken glass, 12,5cm, sterile
Green Z ® Powder	For the absorption of liquids
Disposal box	For collecting contaminated glass fragments

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Further information: Berner International GmbH